PuroOxi

PURO-SPHER

**Concentrated Surface Disinfectant
(Intended for Fog Machines)**

Bactericidal - Virucide - Fungicide - Deodorizer

Immediate and Persistent Activity

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1L 5L

PURO-SPHER

Concentrated Surface Disinfectant
(Intended for Fog Machines)

Product information:

*Puro-Spher is a concentrated disinfectant used through wiping, spraying or fumigation.

*Puro-Spher provides a wide anti-microbial activity against bacteria, spores, viruses and fungi, as well as a residual action allowing continuous disinfection after use.

Recommended Uses:

-PURO-SPHER is a concentrated high level disinfectant used through fumigation, wiping or spraying. -For terminal or turn over disinfection in all critical and non-critical areas in health care facilities such as operating rooms, intensive care and emergency units, isolation rooms, schools and colleges, warship places warehouses ... etc.

Instructions For Use:

*Shake well before use.

*Dilution: 1:3 by adding 1L from Puro-Spher to 3L of water.

*With fogging machines:

-Surfaces must be properly cleaned and rinsed prior to fumigation. -Use a fumigator nozzle of 7-15 microns. -Fill the fumigator with the prepared solution. -Turn off the ventilation and conditioning systems, and close all windows and doors. -Allow about 1 m distance from surfaces and start the fumigation procedure around corners and surfaces, from top to bottom and side to side for a sufficient period to cover all areas. -Fog into drawers, cabinets and furniture when possible. - when procedures are done, wait for 5 minutes then turn on the ventilation system for proper aeration and for the area to dry before re-using it again. *With wiping or moistening: wipe evenly on pre-cleaned and rinsed surfaces. Allow to dry.

Composition:

ADBAC blend (0.3%), DDAC (0.2%), Rust inhibitor (0.1%), Catalysts (0.4%), Inert ingredients (99%).

Anti-Microbial Efficacy*:

Bactericidal: Pseudomonas aeruginosa, Staphylococcus aureus, Chlamydia trachomatis, Enterobacter cloacae, Actinomyces naeslundii, Ureaplasma urealyticum, Streptococcus faecalis & mutans, Lactobacillus leichmanii & paracasei, Enterococcus hirae, Escherichia coli, Coliforms, Klebsiella pneumonia, Listeria monocytogenes, Bacillus subtilis & cereus, MRSA, Salmonella typhi & choleraesuis and Mycobacterium tuberculosis.

Fungicidal including fungal spores: Aspergillus niger & flavus, Candida albicans and Trichophyton mentagrophytes.

Virucidal: HIV, Polio, Herpes simplex, Influenza, Cytomegalovirus, Hepatitis A & B, Rotavirus, Foot & Mouth Disease, Avian Influenza, Norovirus and SARS.

Microbicidal: Dermatophagoides pteronyssinus.

* Tested on the active ingredient blend contained in this product.

WARNING :

Fogging should not be done with person who is suffering or recovering from respiratory diseases. Personal Protective equipment is recommended. Avoid eye contact. Do not ingest. Incompatible with anionic detergents and natural soaps.

First Aid:

In case of skin contact with the concentrate, rinse the area the area thoroughly with water. In case of ingestion, do not induce vomiting, drink a glass of water and seek medical advice. In case of eye contact, flush eye with water.

Redness if any should dissipate within 30 min. If irritation persists consult a physician.

Storage:

Temperature +5°C to 40°C. Store out of reach of direct sunlight. Keep out of reach of children.

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Manufactured by Sadara group - Lebanon