# PuroOxi

## SONIDET CLEANER

Concentrated Cleaner and Disinfectant  
for Medical & Surgical Instruments

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5L

### Product information:

SONIDET CLEANER is a concentrated cleaner and disinfectant that removes organic and microbial loading from instruments prior to sterilization.

SONIDET CLEANER contains corrosion inhibitors to protect instruments from rust.

#### SONIDET CLEANER:

- Effectively dissolves organic soiling and needs no scrubbing.
- Cleans and disinfects in 10 minutes.
- Compatible with metals, glass and rubber.
- Aldehyde, chlorine and phenol free.
- Non-toxic and non-irritant.
- Extends the lifespan of the instruments and the cold sterility.

### Recommended Uses:

- Scrub sensitive instruments and equipment.
- Ultrasonic bath solution.
- Any rust sensitive medical or surgical instrument or surface.
- Cleaning and disinfecting of impression molds and dental prosthetics.

### Instructions For Use:

- \*Rinse instruments with water.
- \*Shake well before use, prepare the solution by adding SONIDET CLEANER at 1:25 means that 40 ml SONIDET CLEANER to 1L of water.
- \*Completely soak instruments in the bath containing the prepared diluted solution.
- \*Contact time:
  - 20 minutes for heavily soiled instruments (Stirring the solution may be necessary).
  - 10 minutes for moderately soiled instruments.
- \*Remove the instruments, rinse well and dry.
- \*Discard the used solution.
- Stability of the unused diluted solution is 7 days.

### Composition:

QAC compounds (0.6%), Surfactants (5%), Anticoagulant (2.5%), Rust inhibitor (0.3%), Catalysts (0.3%), inert ingredients (91.3%).

#### Anti-microbial Efficacy\*:

**Bactericidal:** Pseudomonas aeruginosa , Staphylococcus Aureus , Chlamydia trachomatis , Enterobacter cloacae , Actinomycesnaeslundii , Ureaplasmaurealyticum , streptococcus faecalis&mutans , lactobacillus leichmani&paracasei , Enterococcus hirae , Escherichia coli , Coliforms MRSA , Salmonella typhi&choleraesuis , Klebsiellapneumoniae , listeria monocytogenes , Bacllusubtilis& cereus and Mycobacterium tuberculosis.

**Fungicidal** including fungal spores:Aspergillusniger&flavus, Candida albicans and Tirochophytonmentagrophytes.

**Virucidal:**HIV , Polio , Herpes simplex , Influenza , Cytomegalovirus , Hepatitis A&B , rotavirus , Foot & Mouth Disease , Avian influenza Norovirus and SAR.

**Microbiocidal:**Dermatophagoidespteronysinus.

\*Tested on the active ingredient blend contained in this product.

#### Warnings:

SONIDET CLEANER is a concentrated product with alkaline ph, it shouldn't be used by uninformed persons. Avoid eye contact. Do not ingest.  
Incompatible with anionic detergents and natural soap.  
Protective rubber or neoprene gloves are recommended for sensitive skin only when using the concentrate.

#### First Aid:

In case of skin contact with the concentrate, thoroughly rinse the area with water. In case of eye contact, flush eye with water. Redness if any should dissipate within 30 min. If irritation persists, consult a physician. In case of ingestion, do not induce vomiting, wash out mouth, drink a large glass of milk mixed with a raw egg and seek medical advice.

#### Storage:

Temperature: 5+ C to 40+ C. Store out of reach of direct sunlight.  
Keep out of reach of children.

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